

Tinnitus Management in the VA using ReSound Live™ TS: A clinical survey

Robin Donham, Au.D., Michael Piskosz, M.S., Board Certified in Audiology, Rachel Severson

Tinnitus is the most prevalent physical disability reported by service members who participated in military conflict, as well as those who served during peace time (Source: 2009 VA Benefits Report). Tinnitus sound enrichment devices from manufacturers have increased significantly in the last five years, suggesting some manufacturers are adapting and developing technology according to the ever-changing needs of service-members and veterans beyond just amplification. One noted problem is few devices treat both hearing loss and tinnitus in one system and most do not have the flexibility to adapt to the ever-changing sound therapy plan as suggested by the VA accepted Progressive Tinnitus Management Program. Tinnitus tools should include the flexibility to treat tinnitus in a relaxing, calming or soothing manner with dynamic sound enrichment with or without hearing loss (Henry, 2005; Atkinson, Atkinson & Hilgard, 1983; Hallam, Jakes & Hinchcliffe, 1998; Tyler & Baker, 1983). ReSound's Live™ TS combination device provides the flexibility to treat hearing loss and tinnitus independently or in a combination approach with premium hearing aid technology. The Tinnitus Sound Generator (TSG) employed in ReSound's Live™ TS combination device provides steady-state broad-band white noise, in addition to more dynamic adjustments for sound enrichment. The TSG features are organized in a simple, flexible and intuitive manner ensuring this tinnitus tool's efficiency during sound therapy sessions.

The purpose of this paper is to present the data from interviews of VA Audiologists who have used Live™ TS since being brought on contract in November 2009.

Key topics include:

- Are the Live™ TS features both beneficial and easy to use considering the nature of busy VA clinics?
- What features in the Live™ TS are proving beneficial?
- How should the Live™ TS devices be used?
- How does Live™ TS compare to other tinnitus tools or devices?

Providing evidence-based tinnitus tools for use in VA clinics when managing veterans with tinnitus is critical. Live™ TS should be used as part of an ongoing,

patient-focused tinnitus program for those suffering from tinnitus with or without hearing loss.

Purpose

"...to reduce tinnitus, perhaps hearing aids should have the option to produce a neutral sound that could be incorporated into the fitting." (Buck, et al., 2006).

The goals of the Live™ TS devices are to provide premium hearing aid technology & a fully functional and flexible tinnitus solution, in a cosmetically appealing and customizable solution to assist in the management of tinnitus and/or hearing loss, helping to provide more relief and a better quality of life.

Background

ReSound Live™ TS provides exceptional hearing technology with the flexibility of a tinnitus sound therapy system.

ReSound's Live™ TS premium hearing technology harnesses all key components of high-end, evidence-based devices with a focus on clarity in any environment without compromising sound quality or comfort. Many studies have been published on end-user's preference for ReSound's Warp™ processing, NoiseTracker™ II noise reduction and Natural Directionality™ II with Surround Sound™ features (Kochkin & Tyler, 2008, Surr, et al 1985 & 1999).

It should be noted that hearing aids alone have been used in tinnitus treatment programs with success (Surr et al., 1985; Surr et al., 1999; Sanchez & Stephens, 2000).

Treatment of hearing loss and tinnitus using the Live™ TS as a combination device can assist in reducing the perception of tinnitus by applying an adjustable white noise, which can help a person learn to ignore or habituate to the sound of their tinnitus. Live™ TS is a hearing aid worn during the day that will also serve to improve communication, while providing sound therapy. Live™ TS models incorporate an additional Tinnitus Sound Generator screen in the Aventa Software, where activation and adjustment of the white-noise generator

ReSound

rediscover hearing

occurs; this feature defaults to “off” until activated by the clinician. This enables the audiologist to determine when and how to use this type of sound enrichment.

The sound generator in Live™ TS relies on an ear-level white noise generator to help reduce the contrast of the tinnitus to the background environment, helping the brain to learn to habituate to the tinnitus or the patient learning to ignore the tinnitus. Research suggests that masker-only devices will not only be difficult to set in treating tinnitus depending on the degree or volume of the perceived tinnitus, but additionally the effectiveness of true maskers will decrease over time (Dineen, Doyle, Bench, et al, 1999, Erlandsson, Ringdahl, Hutchins & Carlsson, 1987; Hiller & Haerkotter, 2005; Jakes, Hallam, McKenna & Hinchcliffe, 1992). True Sound Therapy, especially when used as part of a Tinnitus Retraining Program (including a counseling component) has not only been proven successful, but some believe that this type of therapy eventually eliminates the need for a noise-generating device altogether (Jastreboff, Gray, Gold, 1996; Schleuning & Johnson, 1997).

Additionally, exclusive to ReSound, is the Live™ TS Environmental Steering Feature, which will decrease the sound generator volume automatically when noise or speech is present and return to the pre-set sound generator levels when the patient is in quiet or soft speech environments. With flexibility as the motivation behind ReSound’s Live™ TS technology, the Tinnitus Sound Generator should be considered a viable option.

Methods

Sixty-one interviews were conducted with dispensing Audiologists in VA clinics. Audiologists were randomly selected from ReSound’s database with only one prerequisite: that he or she has ordered the Live™ TS devices since November, 1, 2009. Survey sample was identified and evaluated by account managers and reported to be experienced with treating tinnitus and trained on appropriate use of Live™ TS device fitting guidelines. Data collection took place from September 1st through November 24th, 2010. Complete confidentiality was maintained by first confirming fitting audiologist according to serial numbers of Live™ TS devices, then removing dispensing audiologist information from interview data and assigning data to a numerical region. Population fit with Live™ TS devices by audiologists was veterans from any conflict with age ranging from 24 years to 88 years, predominately male. This follows typical VA trends.

Interviews were conducted over the phone by inside account managers trained on Live™ TS features.

Questions were posed on a ten-point scale, however, optional comments following each question were encouraged by interviewer. If unlisted responses were provided by the responder, the category options were updated to ensure complete option list and exclude any biases on the part of the interviewee. Unlimited time was given for answering each question maximizing the validity and thoroughness of the responses.

Results



Figure 1: Average scores of different measured parameters of Live™TS.

VA TS Survey		
Live™TS compared to other tinnitus products	Percent	# Responses
Worse	0%	0
Same	8%	5
Much Better/Better	62%	32
Hasn't Tried Others	25%	15

Figure 2: Live™TS compared to other tinnitus available tinnitus products.

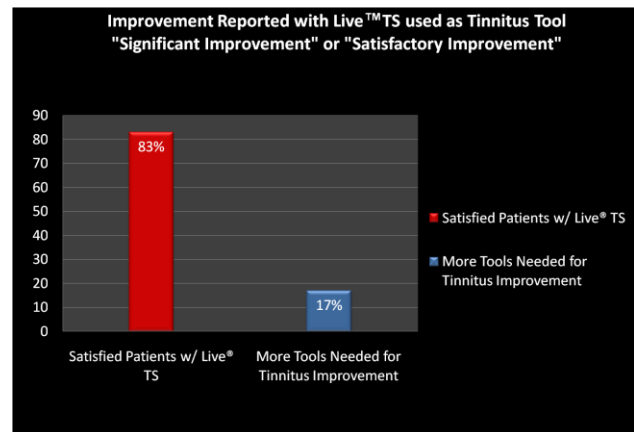


Figure 3: Improvement reported using Live™TS.

Conclusion

Evidence from the survey demonstrated the effectiveness of the Live™TS device in helping clinicians achieve their tinnitus treatment goals, and provides the flexibility to meet individual patient's needs with a 'significant or satisfactory' improvement found in 83% of respondents who activated the Sound Generator in tinnitus fittings.

Since 2007 tinnitus has increased more dramatically than hearing loss and is now the number one service connected disability in veterans. The research and statistics suggest that Live™TS is an efficient, effective, flexible and therefore viable option for use in VA clinics as a tinnitus tool. ReSound's Live™TS is built with the flexibility and advanced technology to accomplish treating hearing loss and tinnitus in just one system, providing many conveniences. Capable tinnitus devices and tools are just one part of an effective, multidisciplinary approach to tinnitus management which should also include productive counseling support techniques and tinnitus education, such as Progressive Tinnitus Management, where a structured program using patient education, sound enrichment/therapy and teaching coping techniques is used to manage reactions to tinnitus. With PTM the patient is encouraged to use whatever sound works to manage reactions to tinnitus and is part of the process to develop a "sound therapy toolbox," which can continuously be updated through categories of sounds. Each sound enrichment plan is customized to address patient's individual needs (Henry, et al., 2005). Utilization of complete evidence-based programs, such as PTM may even further improve the success of tinnitus tools such as Live™TS.

According to this Sound Classification chart, Live™TS falls into at least two groups:

1. *Soothing*: Feel instantly better when this type of sound is heard
2. *Background*: Neutral sound; easy to ignore, but provides noise to defocus from tinnitus
3. *Interesting*: Completely controls or shifts auditory attention from tinnitus to sound

	Environmental	Music	Speech
Soothing	Rain	Canon in D	Kids playing
Background	White noise	High School Band	Boring Lecture Sports Radio
Interesting	Football Game	Child's First Song	Toddler's reason for not sleeping

Figure 4: Classification of sounds using PTM. Live™TS falls into two of these categories.

The data suggest ReSound Live™TS can help provide significant improvement for veterans suffering from tinnitus. Live™TS offers the flexibility to address your patient's needs, for both hearing loss and tinnitus treatment. Future studies need to incorporate the type of counseling or tinnitus management being performed in conjunction with the Live™TS tinnitus tool, to better understand the interaction between the two.

References

- Atkinson, R. L., Atkinson, R. C., Hilgard, E. R. (1983). *Introduction to Psychology*, 8th Edition, 640. New York, Harcourt, Brace, Jovanovich.
- Buck, T.F., George, K.L., Turner, J.G., Verlinden, N.H.P. (May 2006). *Content Suggestions for Universally Designed Hearing Aids*. Worcester Polytechnic Instituted 1865.
- Dineen, R., Doyle, J., Bench, J., Perry, A. (1999). The influence of training on tinnitus perception: an evaluation 12 months after tinnitus management training. *British Journal of Audiology*, 33, 29–51.
- Erlandsson, S., Ringdahl, A., Hutchins, T., Carlsson, S. G. (1987). Treatment of tinnitus: a controlled comparison of masking and placebo. *British J of Aud*, 21, 37–44.
- Hallam, R. S., Jakes, S. C., Hinchcliffe, R. (1988). Cognitive variables in tinnitus annoyance. *British Jof Clinical Psychology*, 27, 213–222.
- Henry, J. A., Schechter, M. A., Loovis, C., Zaugg, T., Kaelin, C., Montero, M. (2005). Clinical management of tinnitus using a 'progressive intervention' approach. *Journal of Rehabilitation Research & Development*, 42, 95–116.
- Henry, J. L., Wilson, P. H. 1999. Cognitive-behavioural therapy for tinnitus-related distress: an experimental evaluation of initial treatment and relapse prevention. In Hazell, J. W. P., (Ed). *Proceedings of the Sixth International Tinnitus Seminar*, 292–296. London, Tinnitus and Hyperacusis Centre.
- Hiller, W., Haerkotter, C. (2005). Does sound stimulation have additive effects on cognitive-behavioural treatment of chronic tinnitus? *Behaviour Research and Therapy*, 43, 595–612.
- Jakes, S. C., Hallam, R. S., McKenna, L., Hinchcliffe, R. (1992). Group cognitive therapy for medical patients: an application to tinnitus. *Cognitive Therapy Research*, 16, 67–82.
- Jastreboff PJ. Gray WC. Gold SL. (1996). Neurophysiological approach to tinnitus patients. *Am J Otol* 17(2):236-40.

- Kochkin & Tyler. (2008). *Effectiveness of Hearing Aids in Mitigating Tinnitus in Typical Patients – Survey of 230 Hearing* Portland, Oregon, American Tinnitus Association.
- Surr, R. K., Montgomery, A. A., Mueller, H. G. (1999). Effect of amplification on tinnitus among new hearing aid users. *Journal of the American Academy of Audiology*, 10, 489–495.
- Schleuning AJ. Johnson RM. Use of masking for tinnitus. *Int Tinnitus J* 1997; 3(1):25-9.
- Tyler, R. S., Baker L. J. (1983). Difficulties experienced by tinnitus sufferers. *Journal of Speech and Hearing Disorders*, 48, 150–154.
- VABenefitsSource:
<http://www.vba.va.gov/REPORTS/abr/2009/abr.pdf>