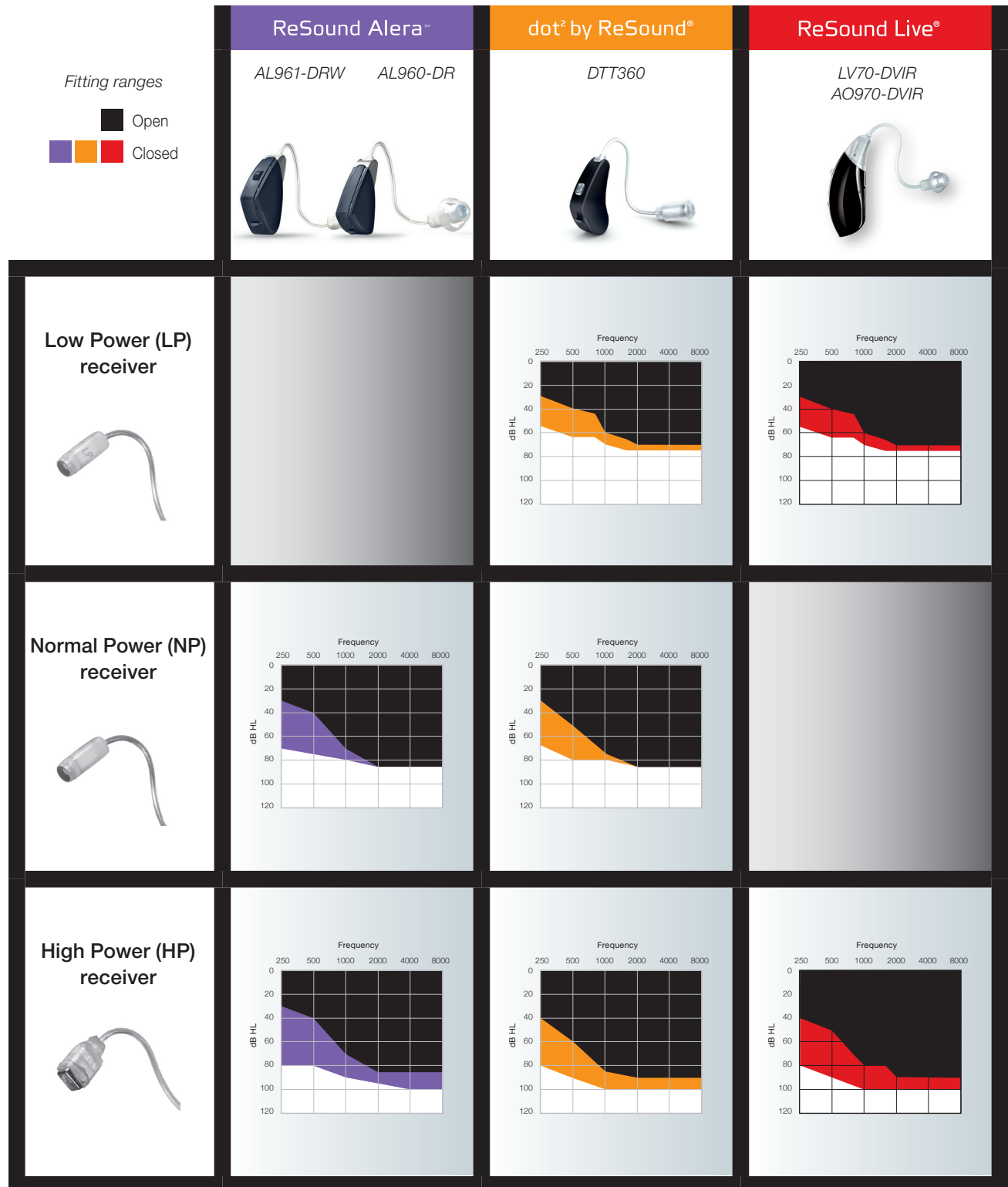
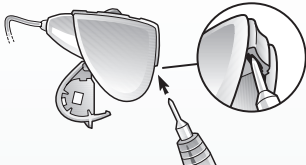
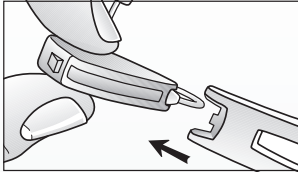
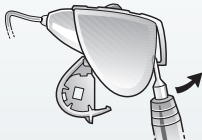
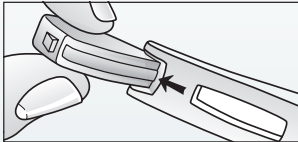
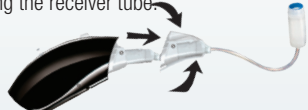
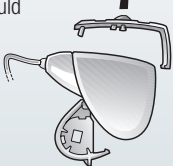
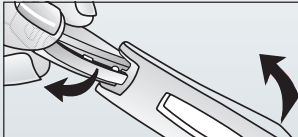
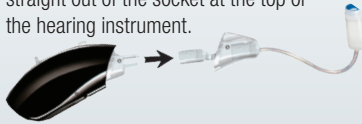
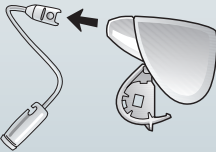
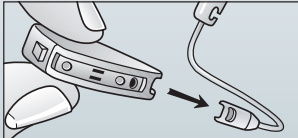

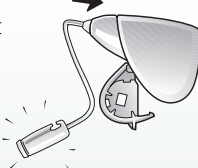
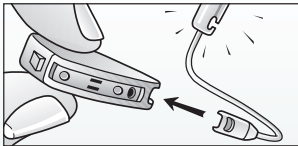

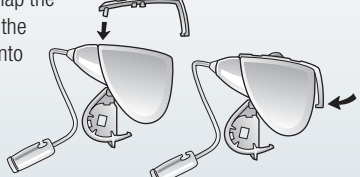
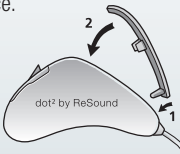
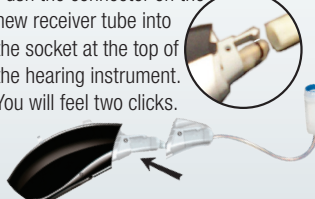
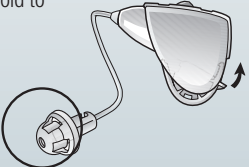
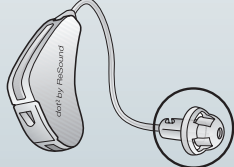
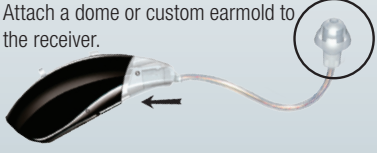


RESOUND RIE (RIC) RECEIVER GUIDE

Multiple receiver options enable ReSound receiver-in-the-ear (RIE) hearing instruments to accommodate a wide range of hearing losses.



Changing the receiver tube

ReSound Alera™	dot ² by ReSound®	ReSound Live®
<p>Open the battery door. Push the long end of the ReSound Alera multi-tool (PN 17327600) into the small opening on the inside wall of the microphone cover.</p> 	<p>Align the narrow end of the dot² by ReSound housing change tool (PN 16352300) with the top of the instrument so that the large prongs straddle the hearing instrument.</p> 	<p>Keep the battery door closed while removing the receiver. The battery door serves to support the outer walls of the housing and eliminate the need for reassembling the housing when the clear collar is removed during receiver installation.</p>
<p>Angle the multi-tool out and away from the hearing instrument to unlock the microphone cover.</p> 	<p>Insert the middle prong of the tool into the seam between the microphone cover and the instrument housing.</p> 	<p>Remove the clear collar by gently rocking it back and forth while pulling it away from the hearing instrument and allowing the collar to slide along the receiver tube.</p> 
<p>Once unlocked, the cover will easily lift off. Care should be taken to avoid crimping the cover during the removal process.</p> 	<p>Tilt the tool back and the microphone cover will pop out. Care should be taken not to lose the cover when it pops out.</p> 	<p>Once the clear collar has been separated from the hearing instrument, gently pull the old receiver tube straight out of the socket at the top of the hearing instrument.</p> 
<p>Pull the old receiver tube straight out of the hearing instrument.</p> 	<p>Pull the old receiver tube straight out of the hearing instrument.</p> 	<p>Separate the collar from the old receiver tube by sliding it over the connector on the end of the receiver tube.</p> 
<p>Insert the new receiver tube by aligning the two pins in the connector with the corresponding port on the instrument, and then pushing the receiver tube in until it is fully seated.</p> 	<p>Insert the new receiver tube by aligning the two pins in the connector with the corresponding port on the instrument, and then pushing the receiver tube in until it is fully seated.</p> 	<p>Thread the connector end of the new receiver tube through the clear collar.</p> 
<p>To replace the microphone cover, insert the tip and then snap the rest of the cover into place.</p> 	<p>To replace the microphone cover, insert the tip and then snap the rest of the cover into place.</p> 	<p>Push the connector on the new receiver tube into the socket at the top of the hearing instrument. You will feel two clicks.</p> 
<p>Close the battery door, and attach a dome or custom earmold to the receiver.</p> 	<p>Attach a dome or custom earmold to the receiver.</p> 	<p>Slide the collar along the tube until it snaps into place at the top of the hearing instrument. Attach a dome or custom earmold to the receiver.</p> 

MK602533 Rev.B

ReSound

rediscover hearing

ReSound GOVERNMENT SERVICES 1-800-392-9932