

**ReSound North America**

8001 Bloomington Freeway  
Bloomington, MN 55420  
1-888-735-4327

[consumerhelp@gnresound.com](mailto:consumerhelp@gnresound.com)

**ReSound Canada**

303 Supertest Road  
Toronto, Ontario M3J 2M4  
1-888-737-6863

ReSound and rediscover hearing are registered trademarks of The GN ReSound Group.  
© 2009 The GN ReSound Group, all rights reserved.

MK602821 Rev.A

# GETTING STARTED *(Quick Guide)*

## ReSound Unite™ Remote Control



**ReSound**


rediscover hearing

## **A** INSTALLING BATTERIES IN YOUR RESOUND UNITE™ REMOTE CONTROL

Remove the battery door, and insert 2 AAAA batteries.

*(Leave the battery door off if continuing on to pair the remote control.)*


NOTE: Please consult the ReSound Unite™ Remote Control User Guide for additional information.

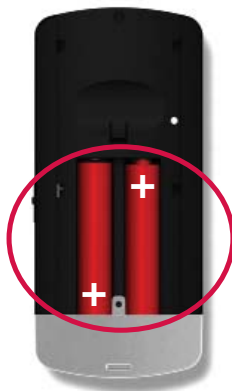

## **B** PAIRING YOUR RESOUND UNITE™ REMOTE CONTROL

In some instances, devices have been pre-paired. Consult your hearing healthcare professional to determine whether you will need to pair your device.


**1.** Open the battery doors on your ReSound Alera hearing instruments.

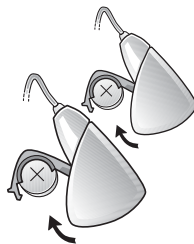


**2.** Press the pairing button on the remote control.



**3.** Close the battery doors on your hearing instruments.

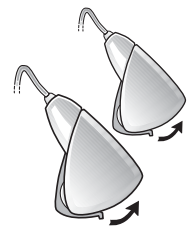

**4.** The remote control will search for the hearing instruments.

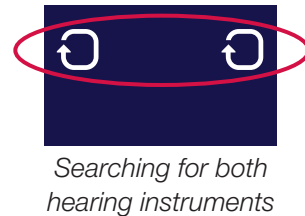

**5.** When pairing is successful, a melody of beeps will be heard in the hearing instrument, the remote control will be ready to use and the current status of the hearing instrument will be displayed.



**6.** Replace the battery door.

