



GETTING STARTED (Quick Guide) ReSound Unite™ Phone Clip

GETTING STARTED (Quick Guide) ReSound Unite™ Phone Clip

Before you can use your ReSound Unite™ Phone Clip it is necessary to:

- A. Charge your ReSound Unite Phone Clip
- B. Pair with your hearing instrument
- C. Pair with your mobile phone
- D. Connect to your mobile phone

Consult your hearing healthcare professional to determine which, if any, of these procedures may have already been completed for you.

MK602895 Rev.A

ReSound and rediscover hearing are registered trademarks of The GN ReSound Group. © 2009 The GN ReSound Group, all rights reserved.

consumerhelp@gnresound.com

ReSound North America
8001 Bloomington Freeway
Bloomington, MN 55420
1-888-735-4327


ReSound Canada
303 Superst Road
Toronto, Ontario M3J 2M4
1-888-737-6863

A CHARGE YOUR RESOUND UNITE™ PHONE CLIP

1. Plug one end of the charging cable into the phone clip.



2. Plug the other end of the charging cable into an electric wall outlet.

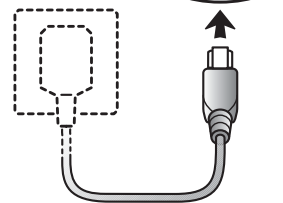


3. Generally, charging is complete when the red LED indicator turns green. However, when charging for the first time allow it to charge for at least 3 hours, even if the LED turns green before 3 hours have elapsed.




B PAIR WITH YOUR HEARING INSTRUMENT

1. Turn on your ReSound Unite Phone Clip by squeezing the multi-function button for 1 second. A solid blue light will display for 4 seconds and then go dark once the phone clip is turned on.

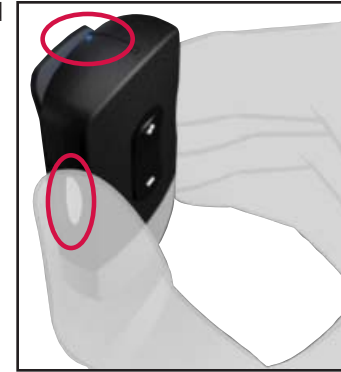

2. Open the battery door on your hearing instruments .

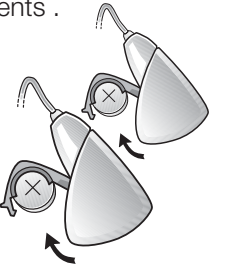

NOTE: Please consult the ReSound Unite™ Phone Clip User Guide for additional information.



B. PAIR WITH YOUR HEARING INSTRUMENT ... continued


3. Remove the cover from the phone clip.



4. Press the pairing button to put the phone clip in pairing mode for 30 seconds.



5. While in pairing mode, turn on the hearing instruments by closing the battery doors on both hearing instruments.

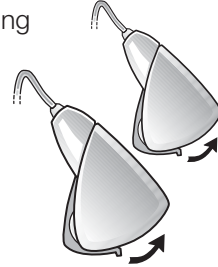


6. A melody of beeps will be heard in both hearing instruments once they successfully pair with the phone clip.

Replace the cover on the phone clip once pairing is complete.

C. PAIR WITH YOUR MOBILE PHONE

1. Turn off your ReSound Unite Phone Clip by pressing and holding the multifunctional button for about 3 seconds until the LED indicator flashes quickly 4 times and then goes dark.

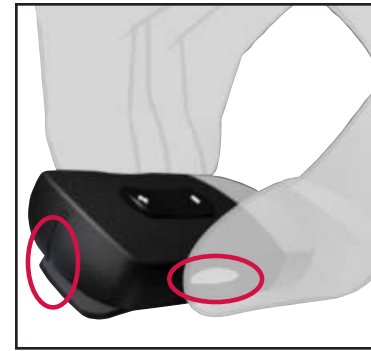

2. Turn on your mobile phone and place it next to the phone clip.




3. Press and hold the multifunctional button on the phone clip for 7 seconds to activate pairing mode. The LED will illuminate and remain solid blue for two minutes while the phone clip is in pairing mode.



NOTE: If you should release the button before pairing mode has been activated, turn the phone clip off and start again.

C. PAIR WITH YOUR MOBILE PHONE ... continued

4. Locate the "Bluetooth" option in the mobile phone menu, and search for Bluetooth™ devices.

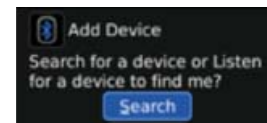


TIP: Ensure that the Bluetooth feature is activated on your mobile phone.

5. From a list of discovered Bluetooth devices, select "Hearing Aid Phone."




6. Enter a passcode of "0000" (four zeros) if prompted.

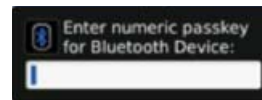

NOTE: Select "Headset" if asked which Bluetooth service you wish to enable.

D. CONNECT WITH YOUR MOBILE PHONE

1. In most instances, the phone will ask you to confirm your intention to connect with the ReSound Unite Phone Clip after successfully pairing.

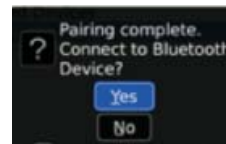

2. If you have to manually connect:

Locate a list of "Paired Devices" in your phones' Bluetooth menu. From that list, highlight "Hearing Aid Phone" and select "Connect."

3. You are now ready to use your ReSound Unite Phone Clip.

TIP: In most instances, a headset symbol on the main screen of your mobile phone will confirm that you have successfully connected.



HELPFUL TIPS:

- Typically, pairing is a procedure that only needs to be conducted as part of the initial setup procedure and does not need to be repeated.
- You may need to repeat the connection procedure if the devices are turned off or moved out of range.
- On some phones you can configure "Automatic Reconnection". Refer to the user guide of your mobile phone for instructions on how to do this.