


ReSound Unite[®]

Pairing Quick Reference Guide

Pairing hearing instruments with the

ReSound Unite[®] Remote Control


- 1.** Open the battery doors of the hearing instruments.

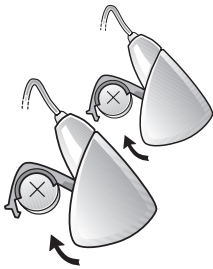

- 2.** Remove the battery door on the remote control, and press the pairing button.



- 3.** Close the battery doors of the hearing instruments.

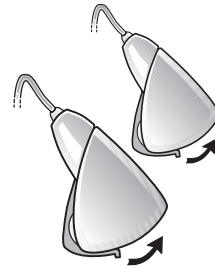

- 4.** The remote control will search for the hearing instruments.



Searching for both hearing instruments

- 5.** When pairing is successful, a melody of beeps will play in both hearing instruments, and the current status of the hearing instruments will be displayed on the ReSound Unite Remote Control.



Program 1 is active

- 6.** Replace the battery door on the remote control.



ReSound


rediscover hearing

Pairing hearing instruments with the ReSound Unite® Phone Clip

1. Turn on the ReSound Unite Phone Clip by squeezing the multi-function button for 1 second. A solid blue light will display for 4 seconds and then go dark once the phone clip is turned on.



2. Open the battery door of the hearing instruments.

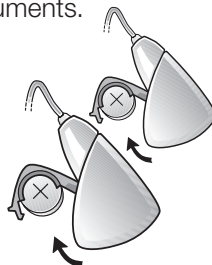


3. Remove the cover from the phone clip.



4. Press the pairing button to put the phone clip in pairing mode. Pairing mode will remain activated for 30 seconds.




5. While in pairing mode, close the battery doors of the hearing instruments.

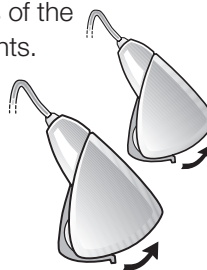


6. A melody of beeps will play in both hearing instruments once they successfully pair with the ReSound Unite Phone Clip. Replace the cover on the phone clip.

Pairing hearing instruments with the ReSound Unite® TV


1. Plug in the ReSound Unite™ TV. The green power light will turn on.

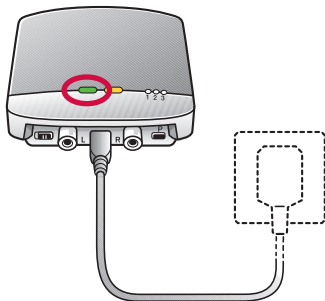


2. Open the battery doors of the hearing instruments.

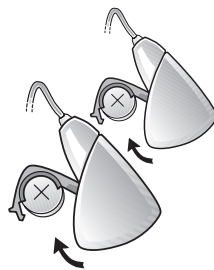

3. Press the pairing button on the ReSound Unite TV. Light #1 will flash and be in pairing mode for 30 seconds.

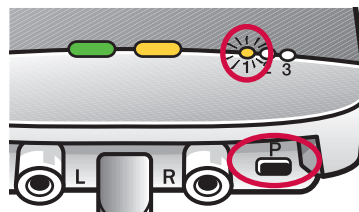

4. Close the battery doors of the hearing instruments while the light on the ReSound Unite TV is still flashing.

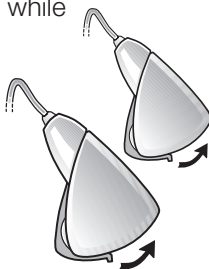

5. A melody of beeps will play in both hearing instruments once they successfully pair with the ReSound Unite TV.

ReSound

rediscover hearing