

ReSound Smart Fit Quick Reference Guide

This guide will help you locate important features as you begin to fit patients. For more detailed instructions, please refer to the ReSound Smart Fit fitting guides in your product manual.

EXPANDED SIDEBAR

The screenshot shows the software interface with the sidebar expanded. Red lines connect text labels on the left to specific UI elements in the sidebar:

- Data logging access:** Points to the 'Data Logging' button at the top of the sidebar.
- Connect hearing instruments:** Points to the 'Right' and 'Left' buttons with ear icons.
- Shows view of hearing instruments associated with this patient:** Points to the visual representation of the hearing aids.
- Mute hearing instruments:** Points to the mute icons on the hearing aid visualizations.
- Media Player:** Points to the 'Media Player' section in the sidebar.
- Collapse sidebar:** Points to the 'Collapse Sidebar' button at the bottom of the sidebar.

PATIENT SCREEN: Profile

The screenshot shows the 'Patient Profile' screen. Red lines connect text labels on the left to specific UI elements on the screen:

- Patient Experience Level:** Points to the 'Patient Experience Level' dropdown menu.
- Initial Programs:** Points to the 'Initial Programs' section, which lists programs like 'All Around', 'Restaurant', and 'None'.
- Fit Patient Button:** Points to the 'Fit Patient' button located in the bottom right corner of the main content area.

FITTING SCREEN: Gain Adjustments

Mute

Display Options

Link/Unlink Hearing Instruments

Gain Level Adjustment

Program Tabs

Gain Adjustment Area

Save

FITTING SCREEN: Advanced Features

Tools Include:

- Reorder Programs
- Recalculate
- AutoRelate
- Manage Program Names
- Copy Program
- Paste Program

Slider to adjust Advanced Features

Advanced Features

Additional Advanced Features


Save

FITTING SCREEN: Environmental Optimizer

Slider to decrease or increase gain for specific environments

FITTING SCREEN: Device Controls


Standard Beeps

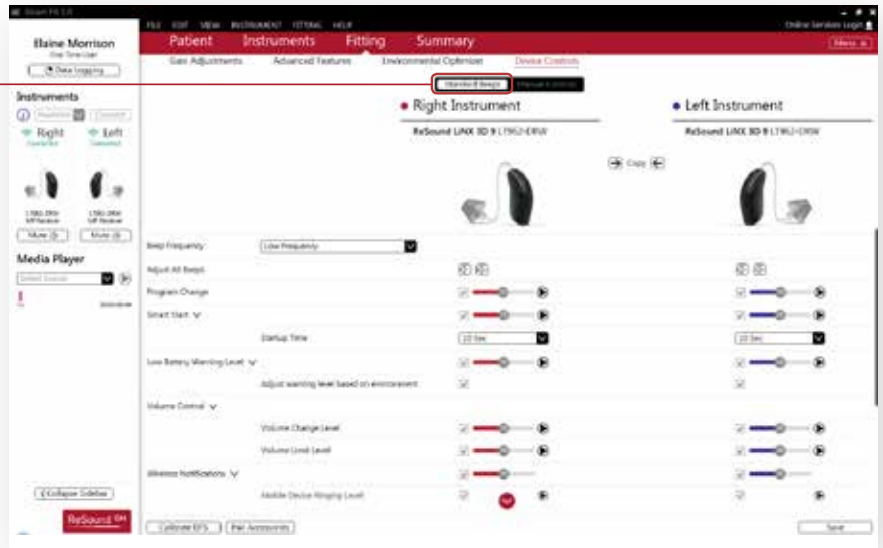

FITTING SCREEN: Device Controls


Manual Controls

Enable/Disable Flight Mode

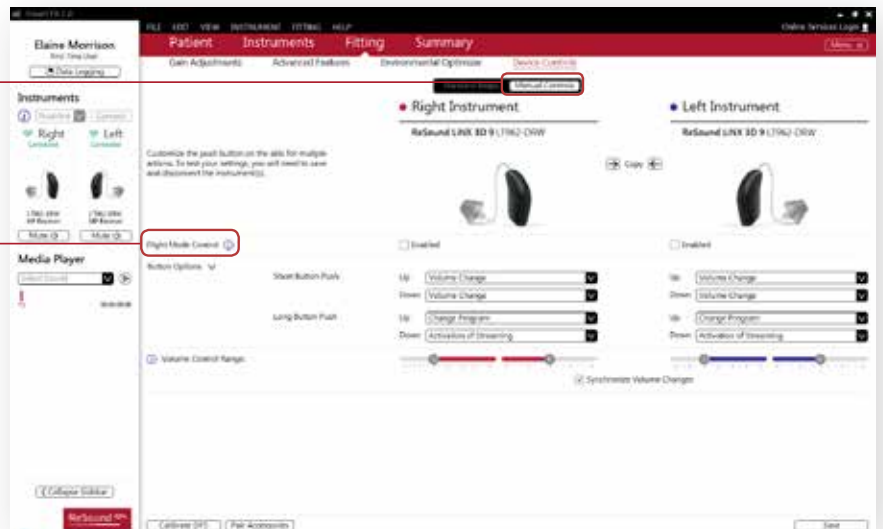


HAMBURGER MENU

Menu Hamburger menu
Houses functions that may not be used during every session

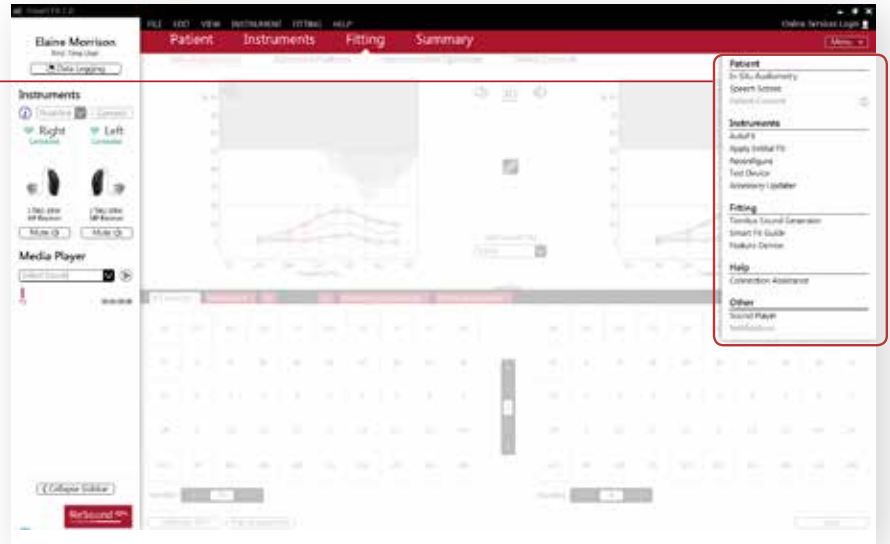

COLLAPSED SIDEBAR

- Head icon – Hover to see patient name and patient experience level
- Pie chart – Click to open data logging
- Red dot by hearing instruments – Not connected
- Green dot by hearing instruments – Connected
- Play icon – Access media player
- Chevron – Expand sidebar

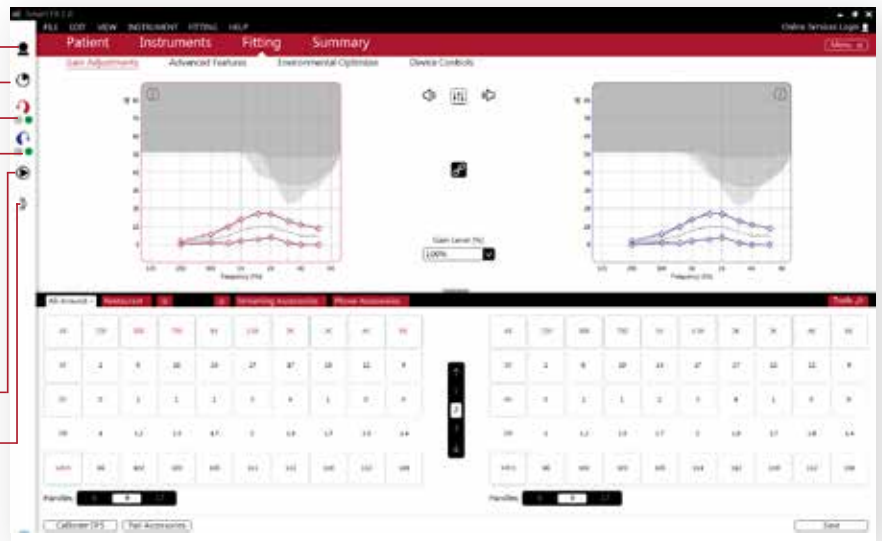

Manufacturer according to FDA:

RESOUND GOVERNMENT SERVICES

8001 Bloomington Freeway
Bloomington, MN 55420
1-800-392-9932
resound.com/governmentservices

